


# DIAQUICK DRUG TESTS

## Product - Overview



*After only 5 minutes, an unambiguous result indicates whether and which drugs are present in the sample.*

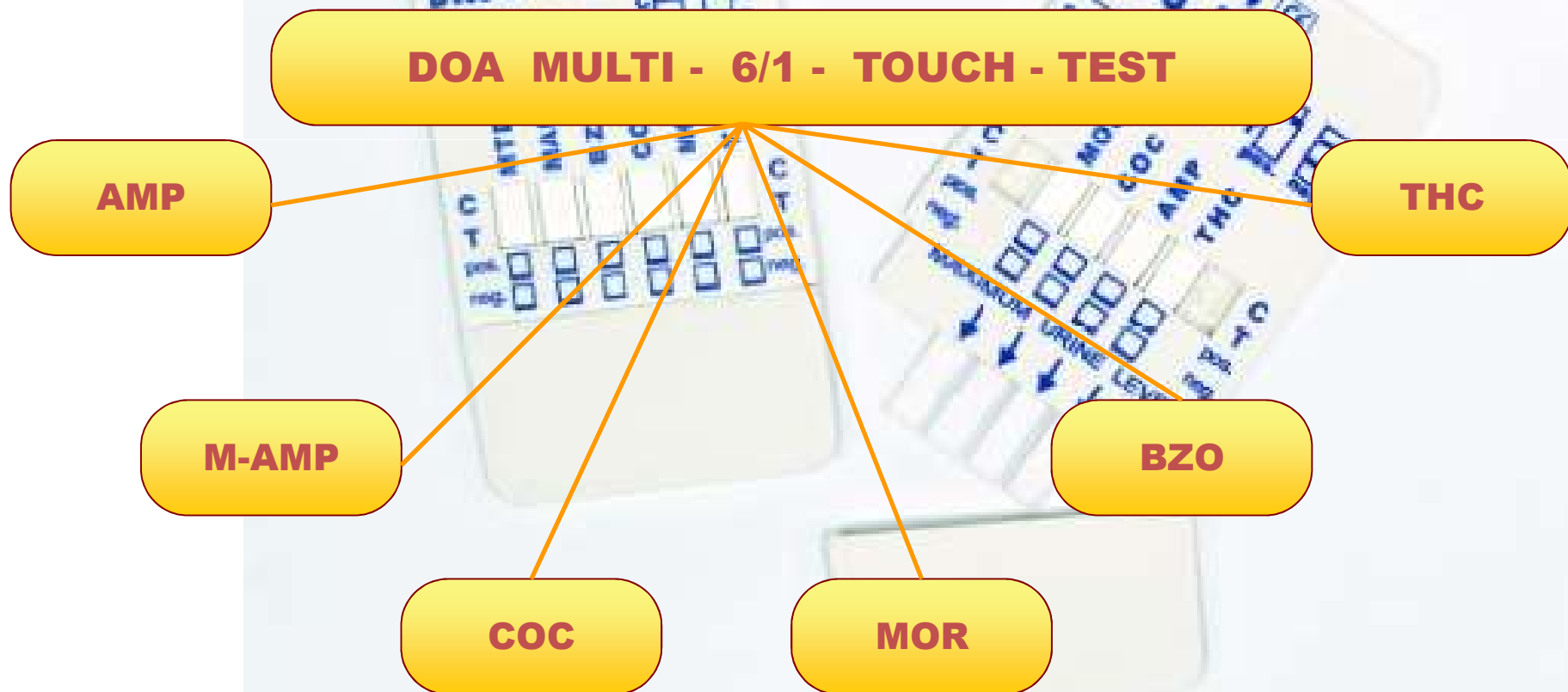

# DOA MULTI TOUCH TEST



## ASSAY PROCEDURE

1



Remove the test device from its protective pouch immediately before performing test.

2



Label the device with identification, the name of the operator performing the test and the date.

3



Remove the protective cap to expose the test adsorbent tips.

4



Draw the adsorbent tips of the DOA Touch Test a few times over the surface (object) to be investigated.

# DOA MULTI TOUCH TEST



## ASSAY PROCEDURE

5



Break the tip off the DOA DEVELOPING SOLUTION TUBE using your thumb.

6



Drop the entire contents of the DOA DEVELOPING SOLUTION TUBE into the protective cap.

7



Replace the protective cap and hold the DIAQUICK DOA MULTI TOUCH TEST upright for 10 Sec.

8



Read the results within 3 to 8 Minutes. Do not read after 8 minutes. Record results immediately after testing.